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Instructions for use

Please read this information carefully because it will help you to use HYLO DUAL® properly.

HYLO DUAL®

Sodium hyaluronate and ectoine

Moisturizes dry eyes, stabilizes the tear film and relieves inflammatory and allergic symptoms such as itching and burning

Sterile, phosphate-free eye drops without preservatives

What is HYLO DUAL®?

HYLO DUAL® is a sterile, phosphate- and preservative-free solution for ocular use.

Composition:

1 ml of HYLO DUAL® contains 0.5 mg sodium hyaluronate, 20 mg ectoine, boric acid, borax and water for injections.

What is HYLO DUAL® used for?

HYLO DUAL® moisturizes cornea and conjunctiva and protects against excessive evaporation of tears. This stabilisation of tears relieves the inflammatory symptoms of irritated eyes. In addition, it also relieves the itching and burning sensations associated with allergy.

When should HYLO DUAL® be used?

Burning, itching, tearing, foreign body sensation or the sensation of dry eyes are symptoms of an irritated eye. Quite often this is a result of insufficient lubrication of the surface of the eye. Other symptoms can be triggered by hypersensitivities against certain substances (e.g. pollen, animal hair or house dust), so called allergies. As an allergy shows similar symptoms it is difficult to differentiate between a dry or an allergic eye.

In these cases, HYLO DUAL® the dual mode of action of sodium hyaluronate and ectoine in HYLO DUAL® helps relieve these symptoms.


HYLO DUAL® contains sodium hyaluronate, a physiologic substance which is found in the eye but also in other tissues of the body. Sodium hyaluronate provides an even, stable, especially long lasting lubrication to the surface of the eye without being rinsed away quickly.

Ectoine is a natural substance gained from microorganisms that live in an extreme environment (e. g. salt lakes). To be protected against these extreme environmental conditions these microorganisms form the natural substance ectoine. Its specific properties are used in HYLO DUAL®. Ectoine increases the binding of water at the cells of the ocular surface. This forms a physiologic barrier on the conjunctiva against allergenic acting substances. Ectoine stabilizes the lipid layer of the tear film that protects against excessive evaporation of tears.

The interaction of sodium hyaluronate and ectoine provides intensive long lasting lubrication of the ocular surface and protects against evaporation of tears. The typical symptoms of itching and burning that occur as inflammatory reactions to environmentally triggered irritations as well as to allergic reactions are relieved by HYLO DUAL®.

How should HYLO DUAL® be applied?


Please do not use HYLO DUAL® if any part of the seal on the folding carton is broken or damaged before using for the first time (☹).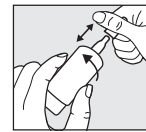

Step 1:

Remove cap before use.

To do this, hold the cap above the bottle. With your other hand, pull away the bottle from the cap with a rotary movement. Before the first application please turn the HYLO DUAL® bottle with its tip down and press the bottom of the bottle repeatedly until the first drop appears at the nozzle. Now the bottle is ready for further use.

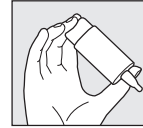
Step 2:

Hold the bottle upside down and place your thumb on the shoulder of the bottle and the other fingers on the base of the bottle.


Step 3:

Support the hand holding the COMOD®-bottle with your other hand as shown.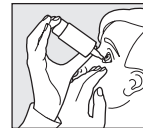

Step 4:

With your head leaning back slightly, gently pull down your lower eyelid. Press the base of the bottle vigorously and quickly. This activates the mechanism to release one drop. Because of the special COMOD® valve system, the size and speed of each drop remain constant regardless of the pressure applied to the base of the bottle. Slowly close your eyes to allow the fluid to spread evenly on the eye surface.

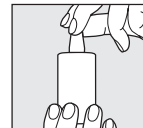Step 5:

Replace cap immediately after use. Ensure that the dropper tip is dry. During use, please avoid any contact between the tip of the bottle and your eye or your skin.

Each bottle releases 10 ml of solution which is equivalent to approximately 300 drops. For technical reasons a small amount will remain in the bottle.

HYLO DUAL® can be used for an unlimited period of time.

HYLO DUAL® is only for you, even people with similar symptoms should not use your eye drops.

What else do you need to know?

If you use any other eye drops there should be at least 30 minutes between the applications, and HYLO DUAL® should always be used last. Eye ointments should, however, always be administered after using HYLO DUAL®.

Because it is preservative-free, HYLO DUAL® is well tolerated even when used over a longer period of time. In very rare cases hypersensitive reactions like burning or excessive tearing have been reported which stopped immediately when the use of HYLO DUAL® was discontinued.

HYLO DUAL® can be used while wearing contact lenses. After inserting contact lenses, waiting for about 30 minutes before applying the eye drops is recommended. In doing so you avoid intolerance reactions which may occur due to interactions between lens care products and HYLO DUAL®.

There are no findings about the use of HYLO DUAL® during pregnancy and lactation. Please ask your doctor or pharmacist before you use HYLO DUAL® if you are pregnant, could be pregnant or are breast-feeding.

How often and for how long can you use HYLO DUAL®?

HYLO DUAL® should be used according to the instructions given by your doctor. Generally, adults and children 6 years and older instil one drop of HYLO DUAL® three times a day into the conjunctival sac of each eye. If you apply HYLO DUAL® more frequently (e.g. more than 10 times per day) or suffer from more severe complaints you should consult an ophthalmologist in regular intervals. HYLO DUAL® is suitable for long-term treatment. However – as in all persisting symptoms – you should consult your ophthalmologist if your complaints do not improve e. g. within two weeks.

When should you not use HYLO DUAL®?

HYLO DUAL® should not be used if you are hypersensitive to one of the ingredients and in children younger than 6 years.

What you need to know about storing HYLO DUAL®?

Do not store above 25 °C (↘^{25 °C}).

HYLO DUAL® can be used for 6 months after opening.

Do not use beyond the expiry date (📅).

Keep out of the reach of children.

The following package sizes are available:

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STERILE A

Medical device

Your opinion about our product is relevant for us!
We are looking forward to get your feedback via mail or
via www.ursapharm.info/hylo dual.

URSAPHARM

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URSAPHARM Arzneimittel GmbH,